INFERUM

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INFERUM

Inferum LLC, Ekaterinburg, Russia

The device for regulative effect for physiological body systems
INFERUM NBP-050

Operating and Service Instruction Manual

We thank you for purchasing the device for physiological body systems INFERUM NBP-050.

Please look through all chapters of the present user manual for efficient and safe application of the device.

This user manual is an integral part of the device and has to be read thoroughly before the device is applied.

CONTENTS

Introduction	21
1. Safety instructions and maintenance	22
2. The purpose	23
3. Application	24
4. Product descriptions	25
5. Technical specifications	26
6. Setting elements	26
7. Contraindications for use	26
8. Conditions and working sequence	27
9. Maintenance	28
10. Order of replacing the power source	28
11. Disposal	30
12. Troubleshooting list	31
13. Manufacturer's warranty	32
Warranty certificate	50
Certificate of compliance	53

INTRODUCTION

The electrostimulation method, applied in INFERUM NBP-050 device gains rising interest because of its efficiency, as well as it gives opportunity to reduce the drug overload and to sustain the health on appropriate level.

The electrostimulation is made by the weak electric current on a threshold of feeling. This method is safe, it helps to normalize vessel walls tone, dilate capillaries, improve hemodynamics of skin microcirculation.

This procedure results in improvement of:

- cenesthesia;
- emotional state;
- efficiency;
- life quality.

The device is a preventive mean and may be applied during course of treatment at home.

Warning!

The device for regulative effect for physiological body systems INFERUM NBP-050 is not a medical device. Do not use the device to set or confirm diagnoses, as well as for treatment of after treatment, medical study, recovery, replacement or change of anatomical organization.

CE Requirements compliance

The present device is fitted with marking CE, conforming the Directive 2004/108/EC Electromagnetic compatibility.

1. SAFETY INSTRUCTIONS AND MAINTENANCE

1.1. Read all the information regarding your safety carefully, as well as recommendations on the cleaning and maintenance of the device.

1.2. The device provides exposure for bioactive points MC-6 (fig.1) on an inner part of wrist and TE-5 (fig.2) on an outer part.

1.3. Do not use the device with other electric device simultaneously to avoid the damage.

1.4. The device contains fragile components. Protect it from shocks.

1.5. The device is not waterproof. Protect it from ingress of moisture. Do not let the product come into contact with rain, moisture or other liquids.

1.6. Do not allow children **under 14 years old** to use the device.

1.7. Keep the device away from heaters; avoid prolonged exposure to direct sunlight at high (over 35°C) temperatures.

1.8. All works on maintenance and repair of the device must be executed by qualified specialists of the manufacturer.

1.9. Use only for its intended purpose as described in this manual.

1.10. **Transportation conditions:** temperature from -50°C to +50°C, relative humidity from 30% to 93%, atmospheric pressure of 70kPa to 106kPa. The device should only be transported in a closed vehicle.

1.11. **Storage conditions:** Packed appliances should be stored indoors at ambient temperatures from -40°C to +50°C and relative humidity of 80% (upper value) at plus 25°C.

1.12. **Operating conditions:** temperature from +10°C to +35°C, relative humidity from 30% to 93%, atmospheric pressure of 70kPa to 106kPa.

Warning!

If the device has been stored at an ambient temperature below 10°C, keep it boxed for three hours at room temperature to avoid condensation.

2. THE PURPOSE

INFERUM NBP-050 device has regulative effect for physiological body systems. It provides fortification of own resources, normalization of physiological body systems and general body condition by course electric noninvasive exposure on biologically active points MC-6 and TE-5, placed on wrists.

The device uses two programs with several phases in each, that differ by working frequency (frequency range from 9 to 77Hz), time and range. These programs provide the cenesthesia improvement for patients with high pressure or patients with low pressure.

Program № 1 provides exposure on the point MC-6, placed on the inner part of a left wrist (see fig. 1) and helps to improve cenesthesia for patient with high pressure. The exposed zone diameter — 10mm. When the icon  flickers white on a device display, the program № 1 is on.

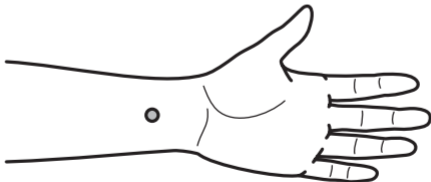
Fig.1. Point MC-6.

Program № 2 provides exposure on the point TE-5, placed on the outer part of a left wrist (see fig. 2) and helps to improve cenesthesia for patient with low pressure. The exposed zone diameter — 10mm. When the icon  flickers white on a device display, the program № 2 is on.

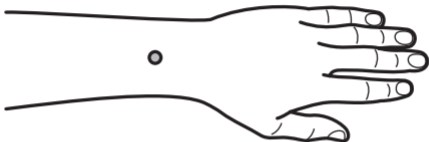
Fig.2. Point TE-5.

3. APPLICATION

3.1. INFERUM NBP-050 device is to be applied as a course exposure on biologically active points, places on wrists. Use the device once or twice per day during 10–15 days. The quantity of repeated courses is individually. Please, control your pressure and use INFERUM NBP-050 as necessary.

3.2. The treatment has a long-lasting effect — the improved cenesthesia turns into permanent after a course of treatment.

Warning!

Drug withdrawal without doctor's consultation is PROHIBITED!!!

Warning!

Use the programs according to the intended purpose.

4. PRODUCT DESCRIPTIONS

The device for regulative effect for physiological body systems IN-FERUM NBP-050 on a figure 3.

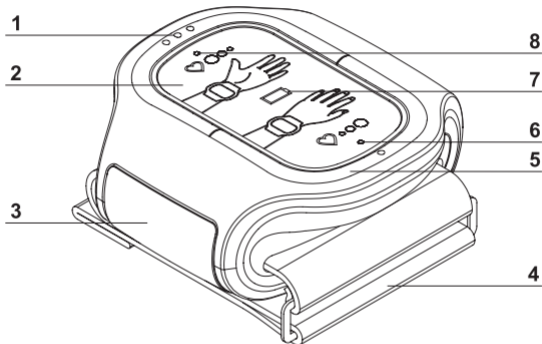
Fig.3

- 1.** Program № 1 start/stop button.
- 2.** Display.
- 3.** Battery chamber cover.
- 4.** Cuff.
- 5.** Program № 2 start/stop button.
- 6.** The indicator of work of Program №2
- 7.** The battery gauge.
- 8.** The indicator of work of Program №1

5. TECHNICAL SPECIFICATIONS

<i>Overall dimension, not more than</i>	<i>.80x80x40 mm.</i>
<i>Weight, not more than</i>	<i>0,3 kg.</i>
<i>Consumption current, not more than</i>	<i>200 mA.</i>
<i>Supply voltage</i>	<i>3±0,6 V.</i>
<i>Power source type AAA</i>	<i>2 batteries.</i>
<i>Index of Protection</i>	<i>IP41.</i>

6. SETTING ELEMENTS

<i>INFERUM NBP-050 device</i>	<i>1pc.</i>
<i>Operating and Service Instruction Manual</i>	<i>1pc.</i>
<i>Consumer container</i>	<i>1pc.</i>

Warning!

Batteries are not available with the set of delivery! Use batteries type LR03/AAA or R03/AAA for this device.

7. CONTRAINDICATIONS FOR USE

Absolute:

- individual intolerance to the electric current;
- implanted cardiostimulator.

Relative:

- epileptic seizure;
- neoplasms of any etiology and localization;
- acute febrility of unclear etiology;
- vein thrombosis;
- condition of acute psychic excitement, alcoholic or drug intoxication.

8. CONDITIONS AND WORKING SEQUENCE

8.1 User should sit or lay in a comfortable position during a treatment with the device.

Warning!

Never conduct treatment in the standing position

8.2 Take off watch or bracelets, roll up a sleeve.

8.3 Choose the exposure point:

Zone MC-6 is zone for users with high pressure;

Zone TE-5 is zone for users with low pressure.

More about the zones see in chapter 2.

8.4. Tighten a cuff of the device to avoid space between cuff and wrist. Make sure that electrodes fit closely to the wrist but not pull it in tight.

8.5. Turn on the device by pushing the proper button:

- Use button 1 (see fig.3), if you have hyposthenic state of health against the background of high pressure. When the icon on a display starts flickering white, the Program № 1 is on. The session lasts — 6 minutes.

- Use button 5 (see fig.3) if you have hyposthenic state of health against the background of low pressure. When the icon on a display starts flickering white, the Program № 2 is on. The session lasts — 9 minutes

8.6. The treatment session consists of several exposure phases, that differ by working frequency, time and range. When the session has been finished the tone signal informs you, the device turns off automatically and the icon is put out. To turn off the device com-

pulsory push and hold button 1 or 5 (according to the exposure program) for not less than 3 sec.

8.7. Take off the device. Have a rest for 20–30 minutes after the treatment.

Warning!

Clean electrodes after every session! If the device is used by several users without cleaning, it may cause allergic reaction or infection. Keep the electrodes dry!

9. MAINTENANCE

9.1. Daily maintenance should include the following procedures:

- examination;
- disinfection.

9.2. Use non-intrusive agents to clean the electrodes (e. g., soapy water or diluted chlorhexidine solution) and soft napless tissues.

9.3. Check the device functioning according to instructions in chapter 8.

9.4. Change batteries if icon  on a display flickers red (chapter 10).

9.5. If you are not going to use the device for a long period, take the batteries out of battery chamber (chapter 10).

10. ORDER OF REPLACING THE POWER SOURCE

Change the batteries as follows if you see flickering icon  on a display or you cannot turn the device on (batteries are discharged):

10.1. Turn off the device if it is turned on.

10.2. Open the battery compartment as it shown on the pic.4.: press on the lock in the direction of arrow 1 and move the lid of the battery compartment in the direction of arrow 2.


10.3. Take out the batteries.

10.4. Leave the device for at least 2 minutes.

10.5. Set new batteries taking polarity into account, as it shown on the pic.5

Warning!

Use only appropriate type of batteries for this device — LR03/AAA or R03/AAA. Use one type of batteries simultaneously and change both batteries at the same time.

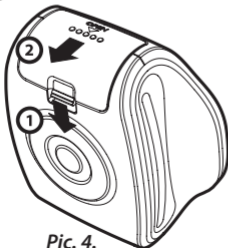Pic. 4.

Pic. 5.

11. DISPOSAL

All packaging materials are not harmful to the environment, they can be reused.

After the end of its working life the device does not present any potential hazardous effect on environment, human life and heat.

Appropriate waste handling of electronic equipment.

Warning!

Waste equipment is not completely unnecessary waste! The device contains valuable materials which can be used repeatedly after utilization with consideration of requirements of environmental protection. They shall be delivered to specially intended for this purpose places (consult with corresponding services in your district) for collection and processing.

12. TROUBLESHOOTING LIST

Problem	Possible causes/ solutions
The device does not start or turns off by itself	Batteries are discharged – change batteries (chapter 10)
The device does not start after the batteries have been changed	Take out batteries. Leave the device for at least 2 minutes, set the batteries (chapter 10)
No perception during exposure.	Dry skin – use a wet cotton pellet to wet the skin
	Electrodes are dirty – clean the electrodes (clause 9.2)

Warning!

All others works on maintenance and repair of the device must be executed by qualified specialists of the manufacturer or service center of the manufacturer

13. MANUFACTURER'S WARRANTY

13.1. The service life of the product is 5 years from the date of manufacture. If the operating instructions are followed properly, the lifetime of the device can exceed the officially established term.

13.2. This product is warranted for the period of 24 months from the original date of purchase.

13.3. The manufacturer or the manufacturer's authorized dealers exclude all liability and responsibility to the customer for any damage occurred after purchase, resulting from:

- mishandling, improper transportation, failure to follow operating and maintenance instructions provided by this manual;
- third party activities;
- force majeure.

13.4. The warranty does not apply to products which have had their warranty seals damaged or removed.

13.5. If, within the warranty period, the product is determined to be defective or incomplete, please send your claims to the manufacturer or the dealer you purchased the product from. Deliver the item back, including a repair request report, with your name, surname, address and telephone number. Enclose a note explaining the symptom or difficulties observed with the product's performance.

Manufacturer's address:

*Inferum LLC,
620100, Russia, Ekaterinburg,
Sibirsky Tract, 12, b. 1, of.206
Tel.: +7 (343) 247-84-51
E-mail: info@inferum.ru
www.inferum.ru*

For product related complains contact our official representative in EU countries.

Address of official EU representative:

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